

Bed Side Wedges Item 469 Fitting Instructions



Lay the Bed Side Wedges across the bed, so that one set will be next to the patient's chest and the other set will be next to their lower legs.

Attach the Velcro Strap Stick to one side of the Side Loops on one of the set of Wedges. Push the Velcro Strap Stick underneath the mattress across to the other side and attach the Side Loops to the opposite side.



Depending on the depth of the mattress, use the Side Loops to make a tight fit under the mattress, so the Wedges will be held firmly in position. Make sure the wedges are equally spaced across the mattress, if necessary they can be moved a little even when fastened to the Velcro Strap Stick.

Repeat with the second set of Bed Side Wedges.



When assisting patients getting in or out of bed, slide the wedge cushions out of the 'Side Compartments' and replace them when the patient is back in bed.

Slide the Wedge Cushions into the 'Side Compartments' with the 90° side facing into the middle of the bed. Note that the label is sewn into the base of the Wedge Cushions and that the cushions are slightly longer than the 'Side Compartments'.



To Remove the Wedges for Washing: Undo the Velcro Strap Stick on both sides of the mattress. The Wedges can then be taken off the bed for washing, leaving the Velcro Strap Sticks under the mattress, ready for refitting. Hand or machine wash up to 80°C. The foam inserts can also be washed. Air or cool tumble dry - do not wring the covers or foam.

Some alternative products to the Bed Side Wedges are; Bed Rail Protectors, Zip-A-Beds, Bed Fall Mats and Stand-Up Alarms check out these on our website.
www.pelicanmanufacturing.com.au