



IMPORTANT NOTICE TO PURCHASER: The following is made in lieu of all warranties, expressed or implied. Seller's and manufacturer's only obligation shall be to replace such quantity of the product proved to be defective. Neither seller nor manufacturer shall be liable for any injury, loss or damage, direct, indirect or consequential arising out of the inability to use this product. Before using, user shall determine the suitability of the product for the intended use, and user assumes all risk and liability whatsoever in connection therewith. No statement or recommendation not contained herein shall have any force or affect unless in an agreement signed by officers of seller and manufacturer. All Pelican Manufacturing P/L products are designed to assist nurses and patients. The patients nursing care should never be neglected, and when Pelican products are in use, continual assessment should be carried out to ensure the products are still appropriate for the patient's condition. Regular maintenance inspection is necessary, and damaged items should be withdrawn from use until rectified. TGA 100909

Refs:documents\is(car (bus) seat pads inst sht 05/10/07

Keep copies of this instruction sheet for Staff Training and with the Patient's Personal Care Plan.

CAR / BUS SEAT SLIDE PADS Item 504B


Copyright © Pelican Manufacturing Pty Ltd

Made in Australia

Temporary Instruction Sheet

Page 1/2

This product is to help people who cannot turn and move into a bus seat near the window. It can also be used to help people getting into the back seat of a car. P.T.O. For car instructions.

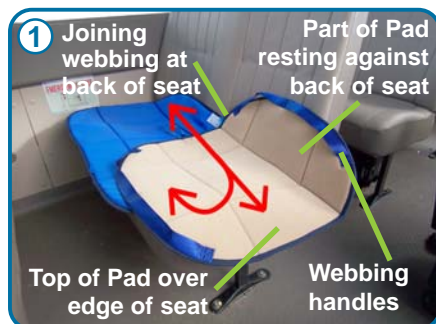

1. Place the two Pads on the seat. The two blue surfaces are against each other and the thin joining webbing is at the back of the seat. The Top Pad is over hanging the side edge of the seat and has part of the Pad resting against the back of the seat, so when the passenger sits down, the webbing handles will be either side of their hips. 2. The passenger has sat down, facing the central aisle of the bus. 3. The passenger holds the back of the seat in front of them, so they turn around to face the front and then slide across to be next to the window.

4. The passenger is now correctly positioned next to the window. Remember to always remove the Pads during the journey. The Pads can be put back in place when getting out of the seat at the destination.

To get out of the seat, reverse the above procedure by the passenger standing up a little so a Carer can position the Pads on the seat again.



If the passenger needs assistance.

5. Carer assisting to turn the passenger to face the front.

6. The Carer supports their own body against the back of the seat. They can hold the webbing handles on the Top Pad, either side of the passenger's hips. It is very important that the passenger holds the back of the seat in front of them and partially stands a little to take some of their weight off the seat.




Hand or machine wash up to 80°C. Air dry or cool tumble dry.

P.T.O. ▶

For Automatic Updates on New Products, email: sales@pelicanmanufacturing.com.au



1. Place the Pads on the seat. The front of the Top Pad should be over the side edge of the seat with part of the Pad resting against the back of the seat.
2. The passenger sits on the Top Pad so the webbing handles are on both sides of their hips.
3. The Carer helps to slide the passenger back into the seat and assist them to put their feet into the car so they face the front.

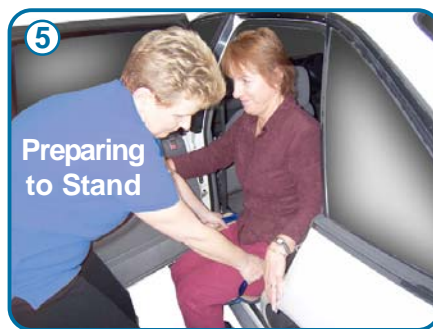

4. The passenger should hold the seat in front of them so they can stand a little, while the Carer removes the Top and Bottom Pads for the journey. Never leave the pads under the person when the car is moving, remember to fasten the car seat belt.

When getting out of the car, the passenger should slightly stand again, so the Carer can place the Top and Bottom Pads back on the seat under them. The Carer can assist the passenger to get their legs out of the car.

5. They can then use the webbing handles on both sides of the passenger's hips, to slide the passenger to the side of the seat.
6. By using the webbing handles the Carer can assist the passenger to stand up. This is easier if the passenger leans forward and supports themselves on the car.

Instruction Guide Only - Before using Manual Handling equipment, obtain instructions from a Manual Handling professional and practise with able bodied people. If the exercise appears to be unsafe, either the Carer/s or the acting patient should stop the exercise and start the manoeuvre again.

Some Other Pelican Products

Handi Soft Turn Pad
 Item 519S - Small, 45cm diam.
 Item 519SL - Large, 51cm diam.
For taller people getting in & out of cars
 Turns 360°
 Very Soft

Car Seat Slide Pad
 Item 504P

Slides in & out of car & turns to face front

Handi-Lift / Walk Belt
 Item 516
 Item 516M (*Miracle Grip*)
To stand & walk
 Standard or Miracle Grip lining

For more information on Car / Bus Seat Slide Pads please visit our website at:
www.pelicanmanufacturing.com.au or email to sales@pelicanmanufacturing.com.au