



# PELICAN™

[sales@pelicanmanufacturing.com.au](mailto:sales@pelicanmanufacturing.com.au) [www.pelicanmanufacturing.com.au](http://www.pelicanmanufacturing.com.au)

5 Ruse Street, Osborne Park, Western Australia 6017  
Free Call 1800 641 577, FreeFax 1800 641 477  
Ph +61 8 9444 4577

ISO 9001  
Reg QEC 5255  
ABN 39 008 771 873



© Pelican Manufacturing Pty Ltd – Permissible for use with Pelican Slings  
S:\Quality Assurance\Master Documents\Sling Inspection Before Use Checklist.doc Revision Status 1

## Sling Inspection Before Use Checklist

This checklist has been developed by Pelican & Slingspector. Ensure this checklist is suitable for your particular sling, sling user, and the environment in which the sling is used.

**Note:** For Slings left under someone, we recommend checking the accessible parts of the Sling, and monitoring the rest of the Sling as it lifts, but check this guidance with your organisation.

	Check attachment compatibility (e.g. webbing loop Sling straps for metal hook hoist, keyhole plate attachments for tilt frame hoists, etc.)
	Check Sling is clean
	Check the label is easily readable, and is able to be understood
	Check the label contains the necessary information to perform the lift safely and correctly, e.g. size, suitability, Safe Working Load (SWL), any special operating instructions or warnings
	Check the condition of the Sling, including the fabric, stitching and any special features such as buckles or Velcro.
	As well as checking the Sling SWL, also check the hoist SWL is appropriate for the person to be lifted
	Check keyhole plates (if fitted) give a good 'click' sound when attached
	Use the Sling with the label on the outside, not next to the person